

enziSurf[®]
HIGH



100% Clean



enziSurf[®] HIGH

Multi-enzymatic foaming spray for high surfaces cleaning in the healthcare sectors (hospitals, nursing and care homes, medical, dental and veterinary practices).

100% Clean

- **MULTI-SURFACE** This multipurpose formula cleans any washable surface without leaving traces. Not corrosive. Compatible with all materials. **FAST** 3 million times per second! The enzymes cut and disintegrate the soil into micro-residues at a breathtaking speed.
- **«DEEP CLEAN»** Efficient and effortless removal of visible and invisible organic soil thanks to the enzymes' action.
- **«PREVENT»** The enzymes anti-deposit effect, prevents soil from re-settling too quickly.
- **«NATURAL»** Ingredients of natural origin: without phosphates, bleach or ammonia. Offers a healthy alternative to conventional cleaning. Fights odors.

Applications

High surfaces:

- Operating table
- Workplan
- Cages
- Carts
- Door handles
- Chairs
- Sinks
- Desks
- Countertops



Product Characteristics

- Deep clean
- Compatible with all materials
- Highly active enzymatic complex
- Ready to use
- Sweet orange fragrance
- Conservation : 24 months after production
- Biodegradability ≥ 95% (OECD 302B)




Instructions for use

Cleaning is an essential infection control step

1. Spray directly on the surface to be cleaned. Spread the product using a clean cloth. Let it work at least **2 minutes**.
2. Met een natte, schone doek veeg het hele behandelde gebied af om de losgemaakte vervuiling te verwijderen.
3. Let dry.

NOTE: make sure to regularly clean the high surfaces in contact with patients. In addition to cleaning, it is recommended to disinfect in order to finalize the decontamination process. Nonetheless, the Superior Health Council recommends using disinfection sparingly in order to fight microbial resistance (AMR).

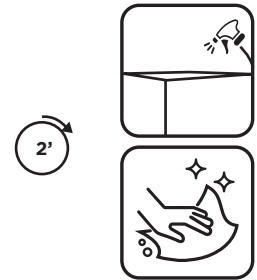

Chemical Properties

Appearance: transparent liquid
white foam when sprayed

Precautions

Store in the original container, closed, between +4°C and +25°C
End of life: 24 months after production

Main Components

Composed of ecocertified natural ingredients

Enzymatic complex:



protease (L)
protease (S)
amylase
lipase

Delivery Units



Natural

Biodegradable

Eco-friendly

Enzymes for patients' safety

Contact:

OneLife s.a.
Avenue Albert Einstein, 15
1348 Louvain-la-Neuve (Belgium)
+32 10 48 34 27 info@onelif-bf.com

onelif-bf.com