

enziSurf<sup>®</sup>  
HIGH



Clean



# enziSurf<sup>®</sup> HIGH

100% Clean

- **MULTI-SURFACE** This multipurpose formula cleans any washable surface without leaving traces. Not corrosive. Compatible with all materials. **FAST** 3 million times per second! The enzymes cut and disintegrate the soil into micro-residues at a breathtaking speed.
- **«DEEP CLEAN»** Efficient and effortless removal of visible and invisible organic soil thanks to the enzymes' action.
- **«PREVENT»** The enzymes anti-deposit effect, prevents soil from re-settling too quickly.
- **«NATURAL»** Ingredients of natural origin: without phosphates, bleach or ammonia. Offers a healthy alternative to conventional cleaning. Fights odors.

## Applications

High surfaces:

- Operating table
- Workplan
- Cages
- Carts
- Door handles
- Chairs
- Sinks
- Desks
- Countertops



## Product Characteristics

- Deep clean
- Compatible with all materials
- Highly active enzymatic complex
- Ready to use
- Sweet orange fragrance
- Conservation : 24 months after production
- Biodegradability ≥ 95% (OECD 302B)



**Multi-enzymatic foaming spray for high surfaces cleaning in the healthcare sectors (hospitals, nursing and care homes, medical, dental and veterinary practices).**

## Instructions for use

Cleaning is an essential infection control step

1. Spray directly on the surface to be cleaned. Spread the product using a clean cloth. Let it work at least **2 minutes**.
  2. Met een natte, schone doek veeg het hele behandelde gebied af om de losgemaakte vervuiling te verwijderen.
  3. Let dry.
- NOTE: make sure to regularly clean the high surfaces in contact with patients. In addition to cleaning, it is recommended to disinfect in order to finalize the decontamination process. Nonetheless, the Superior Health Council recommends using disinfection sparingly in order to fight microbial resistance (AMR).



**Sewage Disposal recommendations:** May be discharged to wastewater treatment installation.  
**Product/Packaging disposal recommendations:** When totally empty, containers are recyclable.

## Chemical Properties

**Appearance:** transparent liquid  
white foam when sprayed

## Main Components\*

Composed of ecocertified natural ingredients

\* Does not contain carcinogens, mutagens, reproductive toxicants or endocrine disruptors.

**Enzymatic complex:**



protease (L)  
protease (S)  
amylase  
lipase

## Precautions

Store in the original container, closed, between +4°C and +25°C  
**End of life:** 24 months after production

## Delivery Units



Natural



Biodegradable



Eco-friendly

Enzymes for patients' safety

Contact:

OneLife s.a.  
Avenue Albert Einstein, 15  
1348 Louvain-la-Neuve (Belgium)  
+32 10 48 34 27 info@onelif-bf.com

onelif-bf.com

Made in Belgium

