



DETECT & DISSOLVE  
BIOFILM MATRIX

# enziSurf<sup>®</sup> INTENSE

• ECO-FRIENDLY • MEDICAL DEVICE •



# enziSurf® INTENSE

**Highly effective multi-enzymatic decontaminant concentrate for a deep cleaning of the critical surfaces** (floors, operating theatres and tables, emergency services, hospital rooms, etc.) in nursing homes, medical, dental and veterinary practices).

- **MULTI-SURFACE** Cleans all surfaces without leaving a trace. Non-corrosive. Compatible with all materials.
- **FAST** Enzymes cut and disintegrate soil ensuring a faster cleaning with less effort.
- **DEEP CLEAN** The enzymes effectively and thoroughly remove visible and invisible organic dirt, even that which seems inaccessible. They also help to get rid of Coronavirus and harmful bacteria. enziSurf® fights odours.
- **PREVENTIVE** The anti-deposition effect of the enzymes prevents dirt from redepositing quickly.
- **NATURAL** enziSurf® offers a healthy alternative to conventional products, protecting the environment, staff and patients thanks to ingredients of natural origin (no phosphates, bleach or ammonia).

## Instructions for use

Cleaning is an essential step in infection control.

1. Dilute the product in **hot water** as follows:  
**2x30ml** in **5L of warm water** or  
**120ml** in **10L of hot water**


Recommended operating temperature between 35-50°C

2. Apply the product to the surface to be decontaminated.
3. Scrub if necessary
4. Let dry.

NOTE: make sure to regularly clean the surfaces in contact with patients. In addition to cleaning, it is recommended to disinfect in order to finalize the decontamination process. Nonetheless, the Superior Health Council recommends using disinfection sparingly in order to fight anti microbial resistance (AMR).


**Sewage Disposal recommendations:** May be discharged to wastewater treatment installation.

**Product/Packaging disposal recommendations:** When totally empty, containers are recyclable

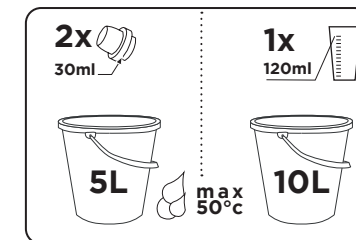

## Applications

- **Critical surfaces**
- Tables
- Operating tables
- Work plan
- Offices
- Counters
- Floors
- Corridors

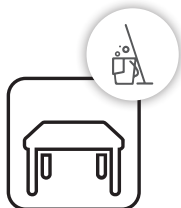

## Product Characteristics

- Highly concentrated formulation combining raw materials of plant origin with enzymes for greater efficiency
- Non-aggressive for the user and the material in contact (compatible with all materials)
- Odourless
- Dye free
- Highly active enzyme complex
- Rate of biodegradability >97% in 28 days (OECD 302B)



## Chemical Properties

**Appearance:** yellowish liquid

## Precautions

Store in the original container, closed, between +4°C and +25°C

**End of life:** 24 months after production

## Main Components\*

99% of the total ingredients are of natural origin.

\* Does not contain carcinogens, mutagens, reproductive toxicants or endocrine disruptors.

**Enzymatic complex:**



protease  
amylase  
lipase

## Delivery Units:



Ref **OL52532**  
6 x 1L



Ref **OL52533**  
1 x 5L



Natural



Biodegradable



Eco-friendly

Enzymes for patients' safety

Contact:

OneLife s.a.  
Avenue Albert Einstein, 15  
1348 Louvain-la-Neuve (Belgium)  
+32 10 48 34 27 info@onlife-bf.com

onlife-bf.com



Made in Belgium